English

YES® WB – water-based personal lubricant

YES® WB is a water-based, discreet, natural silky gel which lubricates and hydrates intimate tissues.

Description

- Alleviates vaginal dryness.
- Natural plant-based intimate hydrating lubricant.
- A soft, clear bio-adhesive gel which adheres to the vaginal walls to lubricate and protect the tissues.
- Lubricates vaginal and penile tissues.
- Natural rubber latex & Polyisoprene condom compatible.
- Matched to vaginal pH (pH 4).
- Guaranteed purity through its certified organic status.
- Reassuringly realistic feel.
- Free of all known skin irritants (no glycerine, no petrochemicals, no parabens).
- Designed to be side-effect free.
- Certified 96% organic by the Soil Association (UK).

Directions for use: YES® WB

YES® WB assists your love making by:

- Enabling penetration.
- Easing vaginal and penile dryness.
- Increasing pleasurable glide and sensation, and promoting the day-to-day comfort of intimate tissues.

May be applied externally and internally, to self and partner. For discreet use, may be applied in advance of intercourse. Use generously as required (make sure you use enough). Re-apply as desired, as YES® WB can be used freely and frequently according to the body’s own needs. This product is a soft clear gel, easy to dispense and distribute, and is designed for finger application, which allows you to decide how much you want to put where. Simply squeeze the soft gel on to your hand and use your finger tips to apply the desired amount to the vaginal tissues. YES® WB may also be used to hydrate the labia. (No applicator included).

Description: YES® WB in applicators

- The discreet & convenient way to deliver lubrication & hydration in the right amount to the right place.
- YES® WB in applicators is designed to be used before love making or at any time to easily enable the distribution of moisture throughout the vagina.
- Pre-filled for single use.

Directions for use: YES® WB in applicators

- Hold the applicator at the wide end and shake the contents down to the narrow end before insertion.
- Twist off the top.
- To apply, insert the narrow end of the applicator only into the vagina. You can lie down, stand or sit to insert the applicator – whichever position is most comfortable.
- Once inserted, squeeze the wide end of the applicator several times to release product high into the vagina.
- When emptied, continue to squeeze the end of the applicator to withdraw it safely. Do not reuse – Risk of Contamination.
- Please dispose of the applicator tube and top by recycling where possible.

Dosage:

- To enhance pleasure and ease in love making: A 5ml applicator is typically sufficient, but more may be used if required.
- For the alleviation of discomfort: A 5ml applicator, used every 2–3 days, is typically sufficient to maintain comfort.

Ingredients

Water, Aloe vera*, Flax extract*, Guar gum*, Locust bean gum*, Xanthan gum, Sodium chlorate, Potassium sorbate, Citric acid, Sodium benzoate.

* Organically grown ingredients.

Protected by International Patents.

Storage: Replace cap when not in use. Store in a cool dry place between 5+30°C. Use before expiry date. Keep out of direct sunlight.

Caution: YES® WB is neither a spermicide nor a contraceptive.

Warning: If the seal on any tube enclosed is damaged or opened DO NOT use, return to place of purchase. If irritation occurs, discontinue use. If symptoms persist seek medical advice. YES® WB is not compatible with Polyurethane condoms.

NB. YES® WB & OB are not suitable for use with the ThinPrep Pap smear test (see website)

YES® OB and YES® OB Vanilla – plant-oil based personal lubricants

YES® OB is a luxurious plant-oil based natural lubricant, whose rich oils and butters nourish and condition skin, giving lasting lubricity and comfort.

Description

- Alleviates vaginal dryness.
- Natural plant-based intimate lubricant.
- For lasting lubrication and glide.
- Dual function: outstanding lubrication and body massage.
- With a velvety sensual feel.
- The natural plant-based alternative to silicone lubricants.
- Safe for use with silicone toys but not latex compatible.
- Plant oils and butters condition skin, no need to wash off.
- Can be applied freely and generously, according to the body’s needs.
- Certified 99.8% organic by the Soil Association (UK).

Directions for use: YES® OB and YES® OB Vanilla

YES® OB assists your love making by enabling penetration, increasing pleasurable glide and sensation. It is a rich, nourishing cream that liquefies into a smooth oil on skin, it is easy to dispense and distribute, and is designed for finger application, which allows you to decide how much you want to put where. Simply squeeze the cream on to your hand, warm it by rubbing between your fingers to ensure a smooth velvety quality, and use your finger tips to apply the desired amount wherever you want it. (No applicator included). YES® OB and YES® OB Vanila are also perfect for massage. Care note: YES® OB may occasionally present with a slightly grainy texture, which is easily remedied. Simply warm the inverted tube in a mug of hot water, so it is completely liquid, shake to mix, then place in the fridge to cool quickly. This will restore the perfect smoothness of the YES® OB texture. This is a function of using exclusively natural ingredients which are temperature sensitive. The product is perfectly safe to use.

Description: YES® OB in applicators

- Enables discreet and hygienic delivery of the right amount of lubrication to the right place.
- Designed to be used before love making or to alleviate discomfort at any time.
- Pre-filled for single use.

Directions for use: YES® OB in applicators

- Hold the applicator at the wide end before insertion. The contents down to the narrow end before insertion.
- Twist off the top.
- To apply, insert the narrow end of the applicator only into the vagina. You can lie down, stand or sit to insert the applicator – whichever position is most comfortable.
- Once inserted, squeeze the wide end of the applicator several times to release product high into the vagina.
- When emptied, continue to squeeze the end of the applicator to withdraw it safely. Do not reuse – Risk of Contamination.
- Please dispose of the applicator tube and top by recycling where possible.

Dosage

- To enhance pleasure and ease in love-making: A 5ml applicator is typically sufficient, but more may be used if required.
- For the alleviation of discomfort: A 5ml applicator, used every 2–3 days, is typically sufficient to maintain comfort.

Ingredients

YES® OB: Sunflower seed oil*, Shea butter*, Sweet almond oil*, Bees wax*, Cocoa seed butter*, Tocopherol (Vitamin E).

YES® OB with Vanilla: Ingredients as above, with Vanilla fruit extract*.

YES® OB in applicators:

Sunflower seed oil*, Coconut oil*, Jojoba seed oil*, Hydrogenated vegetable oil, Grape seed oil*, Tocopherol (Vitamin E).

* Organically grown ingredients.

Protected by International Patents.

Storage: Replace cap when not in use. Store in a cool dry place between 5+30°C. Use before expiry date. Keep out of direct sunlight.

Caution: YES® OB/YES® OB Vanila and YES® OB in applicators are neither a contraceptive nor a spermicide.

Warning: If the seal in any tube enclosed is damaged or opened DO NOT use, return to place of purchase. If irritation occurs, discontinue use. If symptoms persist seek medical advice. YES® OB/YES® OB Vanila/YES® OB in applicators are not compatible with condoms.

Pleasure tip: Experience the Double Glide effect Apply a layer of WB on top of OB for a lasting and new dimension in love making.

Why certified organic?

When you buy something that is certified organic, not only are you making the best choice for your own wellbeing, your purchase is also supporting a system that respects human health, animals, wildlife and the environment. Certified organic products meet stringent standards and organic certification is your assurance that our ingredients and products have been grown and processed according to strict procedures. Production, processing, distribution and sales are monitored to maintain the finest organic and ecological integrity.

Packaging and recycling

YES® aims to minimise the environmental impacts of all our packaging. So please do help us achieve that aim. All our bottles are Polyethylene which is the most widely accepted plastic by recycling centres. The cartons, boxes and accepted inserts can all be recycled.

100% Guarantee: YES® products are 100% satisfaction guaranteed. Please contact us by phone or email if you have any concerns or questions about this guarantee.

Certified Organic by: The Soil Association, Bristol, UK.

Approved by: The Vegetarian Society.

MT Malti

YES® WB – lubrikant personali bbażat fuq l-ilma

YES® WB huwa gell morbidu naturali diskret, ibbażat fuq l-ilma li jilubrika u jidrata t-tessuti intimi.

YES® OB u YES® OB Vanilla – lubrikanti personali bbażati fuq iż-żejt tal-pjanti
YES® OB u OB Vanilla huma lubrikanti naturali lussuz ibbażati fuq żejt tal-pjanti. Iż-żejt u l-butir rikki tagħhom isostnu u jikkondizzjonaw il-gilda, u jaġtu lubrikazzjoni u kumdità dejjema.

YES® OB f'applikaturi – lubrikant personali bbażati fuq iż-żejt tal-pjanti

YES® OB f'applikaturi huma lubrikant naturali lussuz ibbażat fuq iż-żejt tal-pjanti b'żejt tal-geww tal-Indi organiku u goġoba.

* Organically grown ingredients.

Protected by International Patents.

Storage: Replace cap when not in use. Store in a cool dry place between 5+30°C. Use before expiry date. Keep out of direct sunlight.

Caution: YES® OB/YES® OB Vanila and YES® OB in applicators are neither a contraceptive nor a spermicide.

Warning: If the seal in any tube enclosed is damaged or opened DO NOT use, return to place of purchase. If irritation occurs, discontinue use. If symptoms persist seek medical advice. YES® OB/YES® OB Vanila/YES® OB in applicators are not compatible with condoms.

Pleasure tip: Experience the Double Glide effect Apply a layer of WB on top of OB for a lasting and new dimension in love making.

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