English

Please read this leaflet carefully before use.

YES® VM

YES® VM water-based organic Vaginal Moisturising gel uses a patented technology to provide natural, long-lasting moisture to dry vaginal mucosa.

YES® VM is available in 100ml tubes and packs of pre-filled applicators.

•When menopausal estrogen depletion causes itching, burning and irritation of the dry vaginal tissues, YES® VM gives effective, fast and long-lasting relief, and restores intimate comfort.

•YES® VM uses bio-adhesive plant polymers which adhere to the vaginal walls and provide prolonged hydration for up to 3 days.

•Case studies show that YES® VM is fast-acting and long-lasting while replenishing intimate moisture safely and naturally.

Provides symptomatic relief from:

- Vaginal dryness
- Menopausal Vaginal Atrophy
- Itching
- Irritation, burning and soreness
- Dyspareunia.

Temporary vaginal dryness can have various origins: after childbirth, when breastfeeding, and when taking oral contraceptives. Certain anti-depressants and anti-histamines can cause intimate dryness, as can extreme stress.

YES® VM is:

- A soft, clear bio-adhesive gel
- Discreet no smell, taste, colour, stickiness or fallout. Close to the texture and feel of a woman's own lubrication
- pH matched to restore and stabilise the protective acidity of the intimate environment, supporting vaginal health
- Hormone-free
- Free from known mucosal irritants, and ingredients such as glycerine, petroleum products, parabens (hormone mimics) which can disrupt or compromise intimate health
- Gentle and natural, for even the most sensitive conditions
- Guaranteed purity through its certified organic status
- Safe for use with latex and polyisoprene condoms
- Trusted by Health Care Professionals and women to bring soothing, safe and welcome relief to the debilitating symptoms of menopausal dryness
- A non-hormonal natural alternative to synthetic chemical intimate moisturisers.

When cancer treatment (surgery or chemotherapy) has resulted in vaginal dryness, YES® VM is safe for hormone sensitive conditions, and successfully moisturises delicate or traumatised intimate tissues. YES® VM is also suitable for use with dilators, whether or not these are part of a cancer recovery programme.

Directions for use – YES® VM in 100ml tube

This product is a soft clear gel, easy to dispense and distribute, and is designed for finger application, which allows you to decide how much you want to put where. Simply squeeze the soft gel on to your hand and use your finger tips to apply the desired amount to the vaginal tissues. YES® VM may also be used to moisturise the labia. (No applicator included)

YES VM® – frequency of use

A 5ml application used every 2–3 days, is typically sufficient to maintain a moist, comfortable vagina.

Provides intimate moisture for up to three days in typical cases of vaginal atrophy.

Cases of severe dryness will benefit from more frequent use (1 application every 1–2 days) initially until improvement is gained.

If using to alleviate painful sex, apply as required before intercourse and to ensure comfort afterwards.

Iso-tonic*: the iso-tonic/iso-osmotic nature of YES® VM is carefully matched to human tissues.

Why is this balance important? Products which are hyper-tonic (often containing chemicals such as glycerine and glycols) can cause irritation to intimate tissues.

By being iso-tonic/iso-osmotic*, YES® VM protects against such irritation.

*for more information, see website

NB YES® VM is not suitable for use with the ThinPrep Pap smear test (see website)

Ingredients

Water, Flax extract*, Aloe vera*, Locust bean gum*, Guar gum*, Sodium chloride, Xanthan gum, Potassium sorbate, Citric acid, Phenoxethanol.

* Organically grown ingredients. 95% organic. Protected by International Patents.

Storage: Replace cap when not in use. Store in a cool dry place between 5+30°C. Use before expiry date. Keep out of direct sunlight.

Caution: This device is not a contraceptive or a spermicide.

Warning: If the seal on any tube enclosed is damaged or opened DO NOT use, return to place of purchase. If irritation or discomfort occurs, discontinue use and seek medical advice. YES® VM is not compatible with Polyurethane condoms.

Why certified organic?

When you buy something that is certified organic, not only are you making the best choice for your own wellbeing, your purchase is also supporting a system that respects human health, animals, wildlife and the environment. Certified organic products meet stringent standards and organic certification is your assurance that our ingredients and products have been grown and processed according to strict procedures. Production, processing, distribution and sales are monitored to maintain the finest organic and ecological integrity.

Packaging and recycling

YES® aims to minimise the environmental impacts of all our packaging. So please do help us achieve that aim. All our bottles are Polyethylene which is the most is the most widely accepted plastic by recycling centres. The cartons, boxes and package inserts can all be recycled.

100% Guarantee: YES® products are 100% satisfaction guaranteed. Please contact us by phone or email if you have any concerns or questions about this guarantee.


Certified Organic by: The Soil Association, Bristol, UK.

Approved by: The Vegetarian Society.


Directions for use – YES® VM in pre-filled Applicators

When you want to reach higher in the vagina, this product offers you a discreet and convenient way to deliver lubrication and moisture in the right amount to the source of discomfort.


1 Shake down the applicator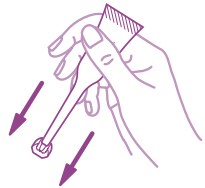

2 Twist off the top

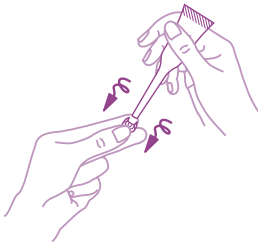3 The applicator can be inserted from three different positions, according to personal ease and preference:

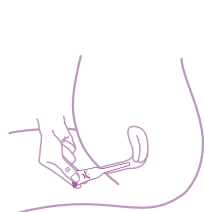
a) How to insert the applicator when sitting


b) How to insert the applicator when standing

c) How to insert the applicator when lying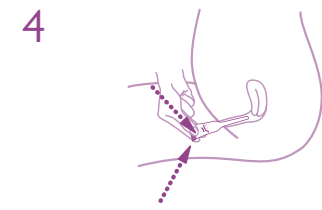

When inserted, squeeze the end of the applicator several times to release product high into the vagina

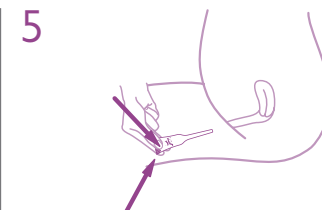When emptied, squeeze the end of the applicator to withdraw it safely. Do not re-use – Risk of Contamination.

Please dispose of the applicator tube and top by recycling

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The YES® bestsellers

See yesyesyes.org for our full collection of natural and organic intimate products

YES® WB

Is a water-based, personal lubricant. This luxurious silky gel is non-sticky, odour and taste-free and feels like natural lubrication. It is pH matched to vaginal pH. Matching vaginal pH may help protect against bacterial and yeast infections. Compatible with natural rubber latex and polyisoprene condoms.

YES® OB

Is a plant-oil based, longer-lasting lubricant and intense hypoallergenic moisturiser and massage oil, which is soothing and richly emollient to nourish delicate intimate tissues. Not condom compatible.

YES® VM

(Vaginal Moisturiser) Is a long-lasting natural vaginal moisturising gel perfect for regular/daily use. It is designed to revitalise and hydrate to relieve vaginal dryness and discomfort. YES® VM is compatible with natural rubber latex and polyisoprene condoms.

YES® CLEANSE

Two natural intimate washes, one rose scented, one fragrance-free, which are sensitively formulated to respect the intimate ecology, with an effective and refreshing foam cleansing action. Each bottle contains over 200 applications.

YES® VM

YES®

YES

Manufactured in Great Britain

The YES YES Company Ltd. The Timber Barn, Selborne Road, Greattham, Liss, Hampshire, GU33 6HG, United Kingdom.

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